


RING ELECTRODES

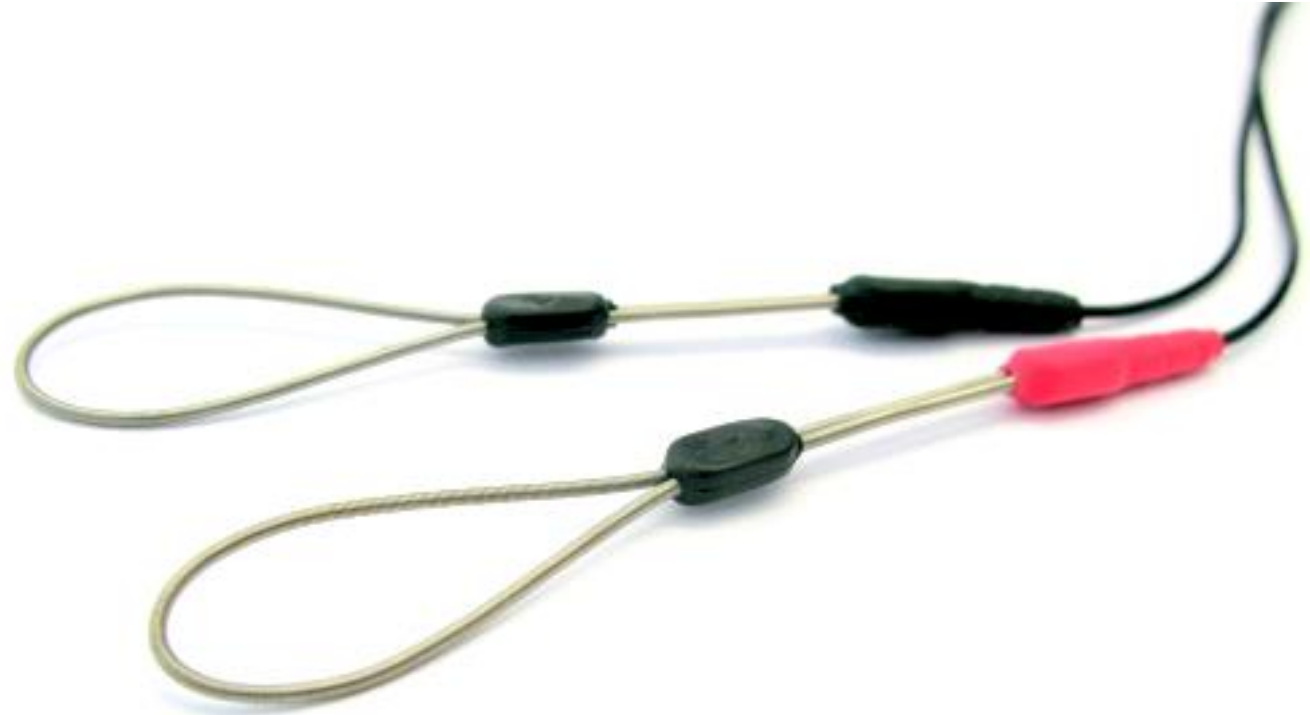

BIONEN S.A.S.

Via P. Petrocchi, 42/1
50127 Firenze – Italia
www.bionen.it

BIONEN
medical devices

GENERAL INFORMATION

Intended use: Electromyography (EMG)

Instruction of use: Fix the electrode where the signal has to be detected. The electrode is foreseen for a repetitive use and therefore can be wasted. Before use, carefully check the integrity of the electrode

Sterilization: NO STERILE. The electrode doesn't need to be sterilized before use. If sterilization is necessary, sterilize with validated processes for ETO or Gamma Ray sterilization.

Cleaning instructions: Carefully **clean** the electrode after each use with water. Leave dry and put the electrode in its plastic pouch, together with the instructions.

Disinfection: Disinfect with validated processes using either Amocid, Lysoformin, Alhyndex, IndicinGG or equivalent products, at high concentration too, without immersing the device into the liquids. Use products according to directives and regulations of the country in which the device is used.

Lifetime: After 5 years from sale, the top shape is not guaranteed.

Packaging: plastic pouch.

Storage: Particular precautions for storing are not necessary but it's recommended to put the electrodes in its box after washing.

Transport and Waste disposal: Any particular transport condition and waste disposal is expected.

Contraindications and interactions: none.

Declared toxicity: none. LATEX FREE

Regulation: Class I device.

Comply with the MDD 93/42/CEE (D.L. 46/97) and manufactured according to the procedures of BIONEN Quality System certified ISO 13485.

RING ELECTRODES

AVAILABLE CONFIGURATION:

REF.	UM	N°	TYPE	L CABLE	MACHINE CONNECTOR	COLOR	DESCRIPTION
0036.001	NR.	1 PAIR	ELASTIC	150 cm	TUKEL 3 CTT	GREY	DIGITAL ADULT
0036.002	NR	1 PAIR	ELASTIC	150 cm	FMM D=1,5mm	GREY	DIGITAL ADULT
0036.003	NR	1 PAIR	NON ELASTIC	150 cm	FMM D=1,5mm	GREY	DIGITAL PEDIATRIC
0036.101	NR	1 PAIR	ELASTIC	150 cm	TUKEL 5 CTT	GREY	DIGITAL ADULT
0036.102	NR	1 PAIR	ELASTIC	150 cm	TUKEL + COCCODRILLO	GREY	DIGITAL ADULT
0036.111	NR	1 PAIR	ELASTIC	150 cm	DRITTO	GREY	DIGITAL ADULT
0036.201	NR	1 PAIR	NON ELASTIC	150 cm	TUKEL 5 CTT	GREY	DIGITAL ADULT
0036.203	NR	1 PAIR	NON ELASTIC	150 cm	FMM D=1,5mm	GREY	DIGITAL ADULT
0036.203	NR	1 PAIR	NON ELASTIC	150 cm	TUKEL 3 CTT	GREY	DIGITAL ADULT
0036.301	NR	1 PAIR	NON ELASTIC	100 cm	FMM D=1,5mm	GREY	PENIENO ADULT
0036.302	NR	1 PAIR	NON ELASTIC	100 cm	FMM D=1,5mm SCHERMATO	GREY	PENIENO ADULT
0036.303	NR	1 PAIR	NON ELASTIC	150 cm	FMM D=1,5mm SCHERMATO	GREY	PENIENO ADULT
0036.304	NR	1 PAIR	NON ELASTIC	100 cm	FMM D=2mm	GREY	PENIENO ADULT
0036.305	NR	1 PAIR	NON ELASTIC	100 cm	FMM D=2mm SCHERMATO	GREY	PENIENO ADULT
0036.306	NR	1 PAIR	NON ELASTIC	150 cm	FMM D=2mm SCHERMATO	GREY	PENIENO ADULT
0036.307	NR	1 PAIR	NON ELASTIC	150 cm	TUKEL 5 CTT	GREY	PENIENO ADULT

